



Installing OSi Software onto i-STAT Alinity v Using a USB Drive




[Click This Link](#)
> Select Download Software




Navigate to Download Your i-STAT Alinity v Handheld Analyzer Software
> Select Downloadable i-STAT Alinity v OSiXX System Software



Save software to a USB drive
** Requires FAT32 formatted USB 2.0 memory stick*
** File must be saved on the top-level directory of the USB drive*

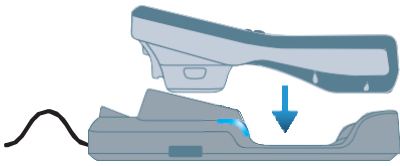

Place instrument into the Base Station and power on the instrument

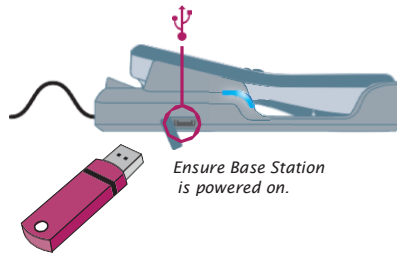

Ensure Base Station is powered on.

Insert USB drive into the USB port in the Base Station



From the Home Screen:
> More Options
> Instrument Options
> Software Installation
> Install from USB



Follow prompts until transfer is complete

SUCCESS



OSi Software Installation Results
SUCCESS

FAILURE



OSi Software Installation Results
FAILURE

If additional assistance is required, please contact Zoetis Diagnostic Technical Support at 1-800-822-2947. Support is available 24 hours/day, 7 days/week.