

# AGID-EIA

Equine Infectious Anemia Virus  
Antibody Test Kit

## THE LINE HAS BEEN CLEARLY DRAWN


### DISTINCT PRECIPITIN LINES FOR CLEARER, EASIER RESULT INTERPRETATION

AGID-EIA from Zoetis is a high-performance agar gel immunodiffusion (AGID) test for the detection of antibodies to equine infectious anemia (EIA) virus.

### RELIABLY CLEAR ANTIGEN-ANTIBODY PRECIPITIN LINES TO DIFFERENTIATE BETWEEN POSITIVE AND NEGATIVE SAMPLES



A  
1-Weak positive  
2-Positive  
3-Positive

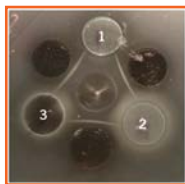

B  
1-Negative  
2-Negative  
3-Negative



C  
1-Positive  
2-Positive  
3-Positive

### COMPARISON STUDY<sup>1</sup>


For two National Veterinary Services Laboratory (NVSL) proficiency panels of samples (n = 39) and of convenience, contemporaneous samples (n = 60) comparing the Zoetis AGID-EIA, the Veterinary Medical Research & Development (VMRD), Equine Infectious Anemia Virus (EIAV) Antibody Test Kit, AGID, and the IDEXX<sup>®</sup> AGID EIA tests:

- The Zoetis AGID-EIA and VMRD tests demonstrated results in agreement with NVSL-reported results
- The Zoetis AGID-EIA and VMRD tests produced identical results, correctly identifying all negative samples
- The majority of IDEXX tests could not be interpreted due to faint precipitin lines
- Results remain unchanged at both 24 and 48 hours for all tests

1. Data on file. Study Report No. D850R-US-18-003, Zoetis Inc.

**DETECT. PREVENT. TREAT.**

**zoetis**

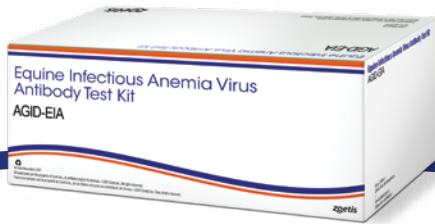

# AGID-EIA

Equine Infectious Anemia Virus  
Antibody Test Kit

## SWITCH TO THE EASY-TO-READ EIA TEST KIT

AGID-EIA uses a highly purified recombinant protein from the EIA virus. A distinct line will form to identify antibodies in serum of EIA infected horses.

No precipitin lines will form if the serum is negative for antibodies to the EIA virus.

### A FEW PRECAUTIONS\*

- Store contents of kit at 36°-45°F (2°-7°C)
- Do not allow reagents to stand at room temperature for excessive periods of time while performing tests
- Handle all reagents and equipment as if capable of transmitting EIA. Burn all containers and all unused contents
- Autoclave all disposable test components and test specimens after use

\*Sale and use of this test is restricted to laboratories by State and Federal (USDA) animal health officials. Positive or questionable results should be sent to NVSL, Ames, Iowa for verification following state regulations. Users should comply with all state or regional regulations regarding handling of any horse with a positive EIA result.

### TRUSTED CUSTOMER SUPPORT

For more information on AGID-EIA or other equine products, visit [www.zoetisus.com](http://www.zoetisus.com).

To order, call Zoetis customer service at **1-888-ZOETIS-1 (963-8471)**.



IDEXX AGID EIA is sold by IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.  
VMRD EIA ELISA is sold by VMRD, Inc. or its affiliates in the United States and/or other countries.

All trademarks are the property of Zoetis Services LLC or a related company or a licensor unless otherwise noted.  
©2018 Zoetis Services LLC. All rights reserved. DIA-00144

**DETECT. PREVENT. TREAT.**

**zoetis**